ClearView® TOTAL Laparoscopic Hysterectomy

LATEX FREE

ClearView TOTAL Laparoscopic hysterectomy

WARNING

- Do not use the ClearView Total if it has been opened or damaged.
- Do not reuse. Discard after one procedure. Function may be impaired through reuse or cleaning.
- The device is difficult to clean after exposure to biological materials and may cause adverse patient reactions if reused. **Clinical Innovations will not be responsible for any direct, incidental, or consequential damages resulting from the reuse of this device.**
- As with all uterine manipulators, a careful clinical examination should be performed prior to use. Certain clinical conditions may present a uterus which is more prone to perforation or bleeding.
- Sound the uterus for depth and direction prior to use. **Do not** use the ClearView Total as a sound.
- Do not use a tip which is longer than the sounded depth. Apply a spacer, or use a shorter-tipped device, so insertion is 1 cm or less the sounded depth.
- Do not use the device without first testing the tip and Occluder balloons.
- The tip balloon is designed to hold up to 10 cc of inflation.
- Do not inflate balloons with air.**
- Use of an incorrect sized ColpoCup could result in patient injury.
- Do not force the tip of the device into a cervical os that is insufficiently dilated.
- A non-inflated, or under-inflated, tip balloon may result in the expulsion of the device from the uterus during use.
- Do not attempt extensive uterine manipulation without a clear laparoscopic view.
- Do not employ extreme or excessive force when manipulating.
- Do not attempt to remove the device by pulling on any exposed tubing. Remove by pulling on the handle only.
- Do not allow prolonged contact between cutting instrument and Colpocup.

INDICATIONS

The Clearview Total is intended for use in laparoscopic procedures where it is desirable to delineate the vaginal fornices, and the surgeon intends to remove or Access intraperitoneal tissue through the vagina by use of a colpotomy or culdotomy incision; such as laparoscopically assisted vaginal hysterectomies or total laparoscopic hysterectomies while maintaining pneumoperitoneum by sealing the vagina while a colpotomy is performed.

CONTRAINDICATIONS




Pregnancy or suspicion of pregnancy, known or suspected uterine or tubal infection, an AID in place, uterine anomaly especially anomalies with a small cavity, an cases in which the surgeon deems it inadvisable or find it difficult to insert.

PRECAUTION

Confirm that the ColpoCup and/or spacer is locked into place prior to starting the procedure.

INSTRUCTIONS FOR USE

- Check the sterile tray for integrity. Remove the device. Using the small syringe (included in package), inflate the tip balloon with sterile saline (10 cc maximum). Remove the syringe and make sure the balloon stays inflated. Reattach the syringe and remove the fluid, deflating the balloon, prior to patient insertion. Using the large syringe included in the package, test the integrity of the Occluder balloon by inflating with 30-60 cc of sterile saline (do not use air); the deflate prior to patient insertion.
- Sound the uterus using the combination Sound/Dilator provided.
- Fort he 7 cm ClearView device; If the uterus sounds to less than 8 cm attach a spacer.
- Fort he 9 cm ClearView device; If the uterus sounds to less than 10 cm, attach a spacer.
- To attach a spacer, with tip balloon deflated, position spacer with the larger-open end facing Towards the pivot and carefully slide the spacer over the tip until it touches the tip support; then Twist the spacer clockwise until it snaps tightly in place (see Figure 1). The manipulator tip should be positioned 1 cm below the maximum sounded depth of the uterus.
- Choose the correct size of ColpoCup to properly fit the patient and then carefully place the ColpoCup over the deflated ClearView Total tip until it touches the tip support or spacer (if used).
- Twist the ColpoCup clockwise until it **snaps tightly in place** (see Figure 2).
- If indicated, dilate the cervix using Standard procedures, or use the combination Sound/Dilator provided.
- Lubricate the distal manipulator tip and balloon. After placing a speculum, slowly insert the tip into the uterine cavity. Completely insert the tip into the uterus until the bottom of the ColpoCup abuts the external cervix. Confirm that the distal end of the Colpocup is secured in the vaginal fornices.
- Using the small syringe, slowly inflate the tip balloon with sterile saline until the base of the ColpoCup is pulled tight against the cervix (10 cc maximum). Continue to hold the plunger of the syringe as it is removed from the inflation valve, to prevent reflux of fluid back into the syringe. Do not over inflate the tip balloon.
- Gently pull on the manipulator to test for proper engagement of the cuff in the uterus. Remove the speculum and rotate the manipulator clockwise to the anteverted position with the manipulator handle between the patient's legs.
- The uterus can now be manipulated by rotating the ClearView Total's handle. Clockwise rotation Elevates the uterus to the anteverted position, counterclockwise rotation retroverts the uterus. The uterus can also be moved laterally by directing the handle to one side or the other.
- Just prior to making the colpotomy incisions, inflate the Occluder balloon with 60-120 cc of sterile saline (do not use air).
- After completion of the procedure, deflate the Occluder balloon and remove the device, in its entirety, from the vagina. **Check to ensure that the device is intact and that no parts remain in the patient.**

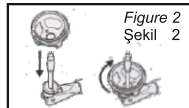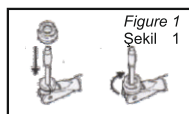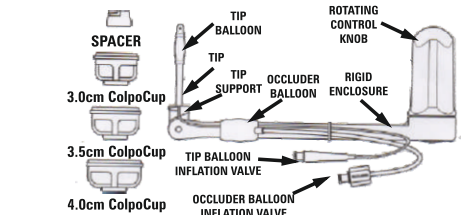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© Copyright 2013