CYPRAMED

C a t a l o g

KULLANMA KLAVUZU VE TALİMATLAR

UYARILAR:

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Is a medical device distributor for GYNECOLOGICAL & UROLOGICAL products such as silicone Pessaries & Vaginal Dilators, Endometrial Sampling Devices, Catheters & Procedure Trays, Cervical Dilators, Aspiration Sets, Laminaria, Biopsy Needles, Leep Electrodes & Accessories and Surgical Instruments

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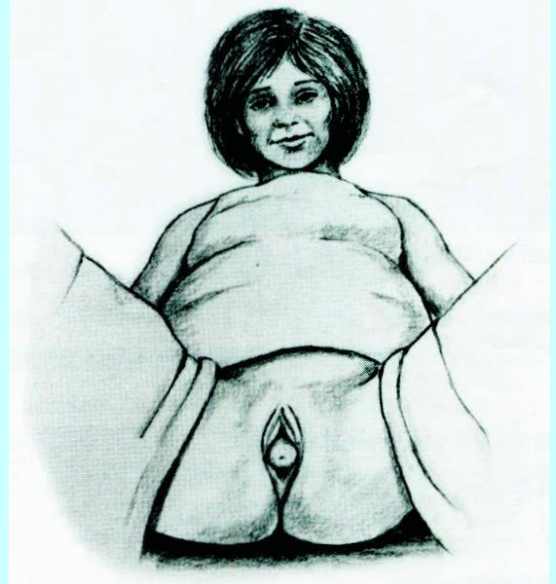
Donut Pessary

DONUT: Support for thirddegree prolapse / procidentia

Position in Pessary

Corrected with Donut pessary

Uncorrected

SILICONE DONUT PESSARY SIZES

DIAMETER inches	DIAMETER mm
2"	50.8
2¼"	57.2
2½"	63.5
2¾"	69.9
3"	76.2
3¼"	82.6
3½"	88.9
3¾"	95.3

SCREENED AREA indicates sizes recommended for office fitting. With these sizes, you will be able to fit 85.2% of patients.

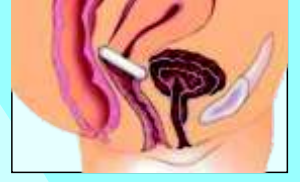
Ring pessary

(INGILIZCE) The Ring pessary, available with or without a supporting membrane, is primarily used for a mild first-degree uterine prolapse or a cystocele.

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SKU(Without Support)	R0	R1	R2	R3	R4	R5	R6	R7	R8	R9
Dimension in inch	1.75"	2.00"	2.25"	2.50"	2.75"	3.00"	3.25"	3.50"	3.75"	4.00"
SKU(With Support)	Rs0	Rs1	Rs2	Rs3	Rs4	Rs5	Rs6	Rs7	Rs8	Rs9
Dimension in inch	1.75"	2.00"	2.25"	2.50"	2.75"	3.00"	3.25"	3.50"	3.75"	4.00"

Cube pessary

(INGILIZCE) The Cube pessary, available with or without drains, may be used to relieve the symptoms of a cystocele or rectocele as well as a second or third-degree uterine prolapse.

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Donut Pessary

(INGILIZCE)The Donut pessary may be used to relieve the stress of a second or third-degree uterine prolapse, a cystocele, or rectocele. This pessary, often used by post-menopausal women, is easily compressed for insertion and removal.

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
Inflatoball Donut

INFLATOBALL : For mild cystocele or rectocele associated with a procidentia/prolapse. The unique design permits individualized fitting and adjustments by varying the amount of air pressure within the pessary. The fully deflated pessary is easy to insert or remove even with a narrow introitus. Inflatoball is sometimes the only pessary the patient can tolerate.

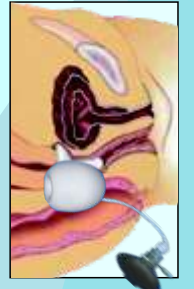SCREENED AREA : indicates sizes recommended for office fitting. With these sizes, you will be able to fit 85.2% of patients. Patient must be taught how to insert, clean and reinsert this pessary daily.

SIZE	DIAMETER	QTY.
S	2"±	
M	2¼±	
L	2½±	
XL	2¾±	

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Gellhorn Pessary

(INGILIZCE) The Gellhorn pessary with drains is used to relieve the symptoms of cystocele or rectocele as well as a second or third-degree uterine prolapse. This pessary's stem with knob is easily folded to facilitate insertion and removal.

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Available Sizes: Gellhorn (GD)/ Short-stem Gellhorn (GS)
GD's stem is 1.5" length. GS's stem is 1.1" length.

SKU(With Drains)	Gd0	Gd1	Gd2	Gd3	Gd4	Gd5	Gd6	Gd7	Gd8
Dimension in inch	1.50"	1.75"	2.00"	2.25"	2.50"	2.75"	3.00"	3.25"	3.50"
SKU(With Drains)	Gs0	Gs1	Gs2	Gs2	Gs4	Gs5	Gs6	Gs7	Gs8
Dimension in inch	1.50"	1.75"	2.00"	2.25"	2.50"	2.75"	3.00"	3.25"	3.50"

Dish Pessary

(INGILIZCE) The Dish pessary, available with or without a supporting membrane, is used to relieve the stress of a second or third-degree uterine prolapse, a cystocele, or rectocele. This pessary may also relieve stress urinary incontinence.

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Oval Pessary

(INGILIZCE) The Oval pessary, available with or without a supporting membrane, is commonly used for a mild first-degree uterine prolapse or a cystocele. This pessary's shape inhibits its rotation in women with a history of vaginal surgery.

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Shaatz Pessary

(INGILIZCE) The Shaatz pessary may relieve the symptoms of a cystocele as well as a second or third-degree uterine prolapse. Its convex shape insures a snug fit while the drainage holes permit easy removal.

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Marland Pessary

(İNGİLİZCE) The Marland pessary, available with or without a supporting membrane, is used to relieve the symptoms of a second or third-degree uterine prolapse, a cystocele, or a rectocele. This pessary's elevated platform may also relieve stress urinary incontinence.

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Gehrung Pessary

(İNGİLİZCE) The malleable Gehrung pessary with a supporting membrane is primarily used for a significant cystocele or rectocele and may also provide relief for a second or third-degree uterine prolapse. This pessary can be manually shaped to accommodate individual problems and anatomies.

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Hodge Pessary

(İNGİLİZCE) The malleable Hodge pessary, available with or without a supporting membrane, is commonly used for a first or mild second-degree uterine prolapse and may also relieve a cystocele. These pessaries, designed to be manually shapeable for individual fitting, are usually prescribed for reproductive-aged women who prefer conservative therapy.

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Ring With Knob

(İNGİLİZCE) The knobbed Ring pessary, available with or without a supporting membrane, has the same indications as the Ring pessary and may also relieve stress urinary incontinence.

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Cup Pessary

The Cup pessary, available with or without a supporting membrane, is used for a first or a mild second-degree uterine prolapse.

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SKU(Without Support)	CP0	CP1	CP2	CP3	CP4	CP5	CP6	CP7
Dimension in mm	50/18	55/23	60/28	65/33	70/38	75/43	80/48	85/52
SKU(With Support)	CPS0	CPS1	CPS2	CPS3	CPS4	CPS5	CPS6	CPS7
Dimension in mm	50/18	55/23	60/28	65/33	70/38	75/43	80/48	85/52

Pessaries may be used to relieve the following symptoms

Pessary Selection Chart

	Prolapse (1-2nd Degree) (2-3 Degree)		Cystocele	Rectocele	Stress Incontinence	Retro Displacement	Incompetent Cervix
Cube		X	X	X			
Cup	X		X		X	X	X
Dish	X	X	X		X		
Donut		X	X	X			
Gellhorn		X	X	X			
Gellhorn (Short Stern)		X	X	X			
Gehrung		X	X	X			
Hodge	X		X		X	X	X
Marland		X	X		X		
Oval	X		X				
Ring	X		X				
Ring w/Knob	X	X	X		X		
Shaatz		X	X				

Fitting Set

(INGILIZCE) Offers a Fitting Set to help determine the proper style and size of pessary for each patient. Packaged in an attractive case, the set includes six popular Ring pessaries and an embossed cross-reference chart to use in selecting the appropriate pessary. One set can be used for multiple patients because the Rings, like all of Panpac's pessaries, can be disinfected and sterilized. Methods for these cleaning processes are included with every pessary.

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Cross Reference Chart Fs1000 Fitting Set Ring

Fit WITH RING	#2	#3	#4	#5	#6	#7
Use						
Gellhorn / Short Stem	G2.00D/S	G2.25 D/S	G2.50 D/S	G2.75 D/S	G3.00 D/S	G3.25D/S
Shaatz	SH2.00	SH2.25	SH2.50	SH2.75	SH3.00	SH3.25
Donut	D2.25	D2.50	D2.75	D3.00	D3.25	D3.50
Ring	R2.25	R2.50	R2.75	R3.00	R3.25	R3.50
Marland	M2.25	M2.50	M2.75	M3.00	M3.25	M3.50
Oval	OV2.75	OV3.00	OV3.25	OV3.50	OV3.75	OV4.00
Dish	DSH60	DSH65	DSH70	DSH75	DSH80	DSH85
Cup	CP60	CP60	CP70	CP75	CP80	CP85
Ring with Knob	RK2.25	RK2.50	RK2.75	RK3.00	RK3.25	RK3.50
Cube	CU41	CU45	CU50	CU56	CU63	CU70

Silicone Vaginal Dilator Sets, Product ID : DT-A, DT-B, DT-C

(INGILIZCE) Each reusable set includes four graduated sizes of dilators used to enlarge the vagina for examination, surgical procedure, or relief of vaginismus.

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Word Bartholin Catheter Set

(INGILIZCE) WC-400MM Silicone Word/Bartholin Catheter w/syringe and scalpel This latex-free Word Bartholin Catheter set contains a silicone inflatable catheter, scalpel and syringe, all needed in the treatment of Bartholin gland abscesses.

(TURKCE) WC-400MM Silicone Word/Bartholin Catheter w/syringe and scalpel

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HSG Catheter Set (LATEX - FREE) With Ballon

(INGILIZCE) Silicone Balloon-Tipped HSG Catheter Set, Latex-Free, available with or without a silicone balloon, are useful in evaluating the causes of abnormal bleeding, menstrual disorders, recurrent miscarriage, and unexpected infertility.

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SKU	Description	Package
BA05F	5 French Catheter	10 Units/Box
BA07F	7 French Catheter	10 Units/Box
BA051F	5 French SureFlex Catheter	10 Units/Box
BA071F	7 French SureFlex Catheter	10 Units/Box
BA300F	HSG Procedure Tray	10 Units/Box

Uterine Manipulator / Injector

(INGILIZCE) UMI4.5 is a simple, safe and effective device for Diagnostic Laparoscopy, Minilaparotomy, Fertility Exams, and Salpingoplasty. It also helps the sealing of the cervical os while providing a fluid or air to the cuff. It is indicated in Diagnostic Laparoscopy, Minilaparotomy, Fertility Exams, and Salpingoplastic procedures where manipulation of the uterus is required.

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Uterine Introducing Catheter (LATEX-FREE Without Balloon)

(INGILIZCE) The UIC Catheter can be used for either sonohysterography or hysterosonography procedure that greatly enhances the analysis of uterine pathology

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SKU	Description	Package
UIC5F	5 French Catheter (without balloon)	25 Units/box
UIC5FA	5 French Catheter (without balloon with Acone)	25 Units/box

Uterine Injector

(INGILIZCE) The Uterine Injectors are double lumen, slightly curved and designed for single use. It was developed to facilitate diagnostic procedures such as Laparoscopy, Minilaps, Salpingoplasties and Fertility Examinations. It is a sterile disposable product consisting of plastic tube, connector and other plastic components which meets the USP recommendation for class VI testing. It is intended to be used during hysterosonography, hydrotubation, hysterosalpingogram, and salpingoplasties

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The Curette

(INGILIZCE) The Curette is used to obtain endometrial tissue lining and superficial layers of the uterine endometrial wall for histological study. Twist & lock syringe with Randall cutting edge

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Preferred Curette

(INGILIZCE) The Preferred Curette and the SureFlex Curette are used to obtain samples of mucosal lining for histological study in cancer screening, endometrial dating, or bacterial culturing.

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Uterine Sound 25/box

(INGILIZCE) The latex-free Uterine Sounds are malleable, disposable, and ready to use for dilating and measuring the cervix. This disposable Uterine Sounds are graduated in centimeters and with the marking from 3 cm, 5 cm, 7 cm, 9 cm thru 11 cm.

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Silicone Vaginal Dilator Sets, Product ID : DT-A, DT-B, DT-C

(İNGİLİZCE) Each reusable set includes four graduated sizes of dilators used to enlarge the vagina for examination, surgical procedure, or relief of vaginismus.

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Word / Bartholin Catheter
(Featuring a silicone catheter)

Inflatable Word/Bartholin catheter is made of medical-grade silicone. It is to be used for the treatment of abscesses of the Bartholin gland usually referred to as a Bartholin cyst. The WC-400 catheter set contains the silicone inflatable catheter with a scalpel, syringe and needle.

Product No:

WC-400 packaged 6 units/bx

Caution:

Federal law requires that a physician order this device.

Procedure:

Make an incision in the outer wall of the cyst to drain the abscess. After draining the abscess, insert the deflated silicone catheter through the incision and into the abscess. Using the 3cc syringe, filled with saline solution, carefully inflate the silicone catheter within the cyst cavity by injecting the saline into the catheter. Once inflated, remove the syringe and needle leaving the self-sealing saline inflated Word/Bartholin catheter within the abscess. After the epithelization of a new orifice is accomplished, the saline solution and catheter are removed from the cyst. After the catheter is removed, the abscess will generally shrink and completely heal leaving the Bartholin gland to function normally.

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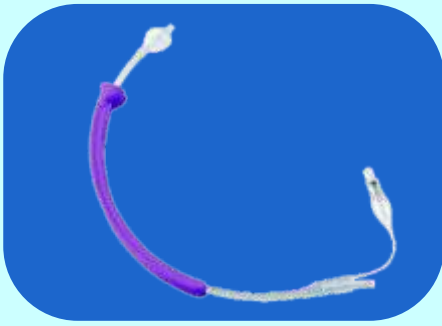
Uterine Manipulator / Injector

Manipulator is a simple, safe and effective device for Diagnostic Laparoscopy, Minilaparotomy, Fertility Exams, and Salpingoplasty. It also helps the sealing of the cervical os while providing a fluid or air to the cuff.

Indications:

Use of the Panpac Manipulator is indicated in Diagnostic Laparoscopy, Minilaparotomy, Fertility Exams, and Salpingoplasty procedures where manipulation of the uterus is required.

Warnings :

- Ensure the catheter is secured to the handle. It's possible that the handle may be loosened during transportation. Snap it back on at the 6cm mark before use.
- Inflate the cuff before manipulation is essential to avoid trauma to the uterus.
- Insert Manipulator along the proper axis to avoid uterine trauma.

Precautions:

- . Test inflatable cuff before insertion for possible leakage.
- . Lubricate catheter tip before use.
- . Check for necessity to dilate cervix before insertion to avoid tearing inflatable cuff.
- . When using any liquid media, be sure to follow manufacturer instruction.
- . After insertion and inflation insure the cuff is properly inflated. (Squeeze pilot balloon and check its tautness.)
- . A deflated cuff may cause injury and trauma to the uterus.
- . Inspect the device for its integrity after use to ensure no parts are left inside.

Adverse Reactions:

Perforation of the uterine wall, Cramping, Infection

Direction for use:

1. Check the sterile pouch has not been opened or damaged.
2. Be sure the catheter is securely snapped in the handle.
3. Test the inflatable cuff by injecting 6 to 7 cc of air into valve (B).
4. After testing, evacuate all air from the cuff for insertion.
5. Sound the uterus for depth and direction, adjust the Manipulator depth accordingly Manipulator is pre-set at 6cm in depth.
6. Ascertain if the cervix needs dilation. The inflatable cuff may tear if the passage is not wide enough.
7. Lubricate the distal tip (C) and cuff (A) for easy insertion.
8. Guide the manipulator carefully along the natural axis of the cervix to avoid injury.
9. Insert the manipulator into the uterus until the face of cervical stop (E) touches the external cervix.
10. Inflate the cuff slowly with 3 to 10 cc of air until the cervical stop being pulled up tightly against the cervix. When detach the syringe, hold the plunger to prevent reflux of air back into the syringe.
11. Pull gently on the manipulator to be sure cuff inflation is adequate. Excessive cuff inflation may cause utero-tubal spasm and physiologic closure of patent tubes.
12. To insure the cuff has not ruptured during procedure, check the tautness of the pilot balloon. A soft balloon indicates a ruptured or leaking cuff.
13. To remove the manipulator, insert a syringe in the valve and deflate the cuff, then remove the manipulator carefully.

Contraindications:

- Pregnancy or suspected pregnancy
- Uterine or tubal infection
- Media allergy
- Gynecological malignancy
- During assisted reproduction technology

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HSG catheter sets can be used for sonohysterography and hysterosalpingography procedures that greatly enhance the analysis of uterine pathology. Using HSG balloon catheters, the procedures are simple and non-traumatic for your patients. A placement sheath eliminates the need for a tenaculum. HSG catheters are designed for single patient use only.

CAUTION

Federal law requires that these devices be ordered by a physician.

INDICATIONS

HSG catheters may be used to evaluate the causes of abnormal uterine bleeding, menstrual disorders, recurrent pregnancy loss or unexplained infertility. These devices are also very useful for the assessment of uterine pathology and assessment of patients on tamoxifen therapy.

CONTRAINDICATIONS

HSG procedures are contraindicated in pregnancies, suspected pregnancies, active pelvic infections, or recent pelvic infections, severe uterine bleeding, gynecologic malignancies, or if the patient is allergic to the contrast media. This device should not be used for intrafallopian procedures.

PROCEDURE

Remove the HSG set from the sterile pouch and remove the balloon protective sleeve from the distal end of the catheter. Using the attached syringe, inflate the silicone balloon by opening the stopcock attached to the syringe luer. Then allow the silicone balloon to completely deflate. This procedure ensures the integrity of the HSG balloon. Attach the media source to the open luer and allow the contrast media to flow through the catheter removing air and clearing the media channel.

DIRECTIONS FOR USE

With the guiding sheath, position the HSG catheter so that the tip of the catheter is at the entrance of the cervix external OS. Gently push the HSG catheter, with balloon, through the cervical canal and into the uterine cavity. Inflate the balloon within the uterine cavity and close the balloon stopcock to prevent the balloon from deflating. Carefully withdraw the catheter until the balloon is resting against the internal OS. Now inject the contrast media into the uterus and proceed with the examination. In cases where the access to the uterus is difficult, the balloon may be inflated within the cervical canal.

The Curette

The ETO Sterilized Curette is used to obtain a sample of differentiated endometrial tissue. The Curette is a 3mm (O.D.) endometrial suction curette with a Randall-like cutting edge at its distal end and packaged with a twist-and-lock syringe. The syringe provides the vacuum or suction during the procedure. The Curette is sterile unless the package is opened or damaged. It is designed for single patient use only.

Product No. : RD003

INDICATION:

The endometrial suction curette- The Curette is a device used to remove samples of materials from the uterus and from the mucosal lining of the uterus by scraping or vacuum suction. These devices obtain tissue samples for purposes of biopsy, precancer screening or they can be used for menstrual extraction.

CAUTION:

Federal law restricts this device to sale by or on the order of a physician.

CONTRAINDICATIONS:

The procedure is contraindicated in suspected pregnancy or in women with acute pelvic inflammatory disease. It is also contraindicated in women with chronic cervical infections or any conditions which contraindicate an outpatient surgical procedure.

PROCEDURES:

While sounding the uterus and using this device, care should be taken to avoid perforation of the uterine wall. Do not depress the syringe's piston while it is in the uterus.

DIRECTIONS FOR USE:

1. Prepare the vagina and cervix as you would for any sterile intrauterine procedure.
2. Expose the cervix using a suitable speculum.
3. Gently insert a uterine sound to determine the depth and direction of the uterine cavity. It may be necessary to grasp the cervix with a tenaculum. If the uterus is anteverted, the grasp should be on the anterior lip of the cervix. If the uterus is retroverted, the grasp should be on the posterior lip of the cervix. Apply gentle traction to straighten any cervical curvature.
4. After the depth of the cervix has been determined, insert the Curette paying close attention to the embossed scale on the Curette sleeve or use the enclosed "stopette." This will provide additional protection from perforation.
5. With the piston of the Curette's syringe completely depressed, the Curette should be inserted and gently passed through the cervical canal and into the cavity of the uterus. Discontinue any traction applied with a tenaculum. Carefully attach the syringe to the base of the Curette. Please note, that the "arrow" at the proximal end of the curette is aligned with the cutting edge or sampling point of the distal end.
6. With one hand holding the proximal end of the Curette, withdraw the piston of the syringe with your free hand creating a vacuum within the Curette. The withdraw motion should be smooth and steady until the piston reaches the end stop of the syringe. Now grasp the piston syringe close to the blue locking mechanism and lock the piston in place by turning the piston 1/4-turn clockwise. Test to be sure the piston is secured in its locked position. When locked, the syringe will provide adequate vacuum or suction during the procedure.
7. After the piston has been retracted and locked to provide a vacuum or suction, rotate the Curette through all four quadrants of the endometrium by using long slow strokes.
8. The Curette should be gently removed from the uterus. Upon examination of the device, you should see a specimen or sample from the endometrium of histological quality. Bleeding is usually minimal, if it occurs at all.
9. Cut off the tip of the Curette just above the sampling point of the device. The sampling point is the entrance port with the Randall-like cutting edge at the distal end of the Curette. Unlock the syringe by turning the syringe piston 1/4 turn counterclockwise and expel the specimen into an appropriate transfer vial.

WARNINGS:

1. In general, any patient with cervical stenosis requires extreme precaution. Do not use force when using this device with these patients. You may use a topical anesthetic prior to the use of the Curette.
2. Be aware of and look for adverse reactions that are occasionally encountered in any intrauterine procedure.

ADVERSE REACTIONS:

1. Patients should be carefully watched for evidence of unusual paleness, nausea, vertigo or weakness. Any cervical manipulation may cause a vasovagal reaction. These symptoms typically subside in about 15 minutes of rest and/or a mild analgesic.
2. In some cases, there may be spot bleeding or mild cramps after this procedure has been performed. The patient should be instructed to notify the physician if spotting continues or if a persistent fever develops.

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San. Tic. Ltd. Şti.